

# Fast Find Grid

## Disposable CT Biopsy Grid

Cat. No. EG6500

### Instructions for Use:

1. Remove backing from tape.
2. Place Fast Find Grid tape side down on the patient, over the area of interest.
3. Scan through the area of interest.
4. On each scan slice, you will see twelve dots on the skin surface.
5. Determine which scan slice the radiologist will want to work from.
6. Have the radiologist determine which dot best represents the point of entry on the patient.
7. You will notice that on each scan slice there is a larger gap between two of the dots. These two dots represent #1 and #2 on the grid.
8. Once the reference dot for the point of entry is chosen, you will need only to count out from either side on the scan slice to determine which number line on the grid the reference dot represents.
9. Move the patient to the pre-determined scan slice location.
10. Turn on the laser light and mark through the area with a felt tip pen where the light crosses the proper line.
11. Before removing the grid, mark a reference point on each corner cross-hair of the grid with a felt tip pen.
12. Remove the grid and perform the biopsy.

**NOTE:** If necessary, the Fast Find Grid can again be placed on the patient by aligning the four reference points marked on the patient with the four corner cross-hairs on the grid.

**CAUTION:** Federal law restricts this device to sale by or on the order of a physician.

Distributed by:



For single use only: Lot 109



Mfg by: Webb Mfg Corp  
1241 Carpenter St.  
Philadelphia PA 19147  
Authorized Agent:  
Cavendish Scott LLC  
P.O. Box 107, Hitchin SG5  
1FW United Kingdom

1 2 3 4 5 6 7 8 9 10 11 12

# Fast Find Grid

## Disposable CT Biopsy Grid



**THE LOW-COST POSITIONING GRID  
FOR QUICK, ACCURATE CT BIOPSIES  
AND DRAINAGE PROCEDURES.**

[WWW.FASTFINDGRID.COM](http://WWW.FASTFINDGRID.COM)